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Align Technology, Inc.
2560 Orchard Parkway
San Jose, CA 95131
USA

Emergo Europe
Molenstraat 15 2513 Bh,
The Hague The Netherlands
+31.70.345.8570

© 2014 Align Technology, Inc.
All rights reserved. D16471 Rev B

vivera
retainers

vivera
retainers

English

Wearing and using your Vivera retainers.

The Vivera Retainer System consists of a series of doctor-prescribed, custom-made, thin, clear, plastic removable orthodontic retainers that maintain and prevent movement of a patient's dentition.

It is important to wear your retainers for the length of time prescribed by your dental practitioner. Here are some tips to help ensure proper use and to avoid damaging your retainers.

Always remember to

1. Rinse new retainers thoroughly with cold water prior to insertion.

Retainer insertion

1. Make sure you have the proper retainer— the upper for the top teeth and the lower for your bottom teeth.
2. To help avoid confusion each retainer is marked with a "U" for upper arch or an "L" for lower arch, in addition to a unique treatment number.
3. You may insert either the upper or lower retainer first. When inserting each retainer, gently push the retainer over your front teeth. Then, apply equal pressure, using your fingertips, to the tops of your left and right molars (back teeth) until the retainer snaps into place. Retainers should fit comfortably and should not be forced into place.
4. To avoid damage to a retainer, do not bite the retainers into place.

Note: If you experience sharp pain or significant discomfort, discontinue use of the retainers and contact your dental practitioner immediately.

Retainer removal

1. Using your fingers, start on one side at the molars (back teeth), and slowly work your way around to the other side.
2. To help prevent damage, avoid unnecessary removal.
3. Avoid use of any sharp object to remove your retainers.
4. Immediately rinse the retainer with water, shake off excess water, and store your retainers in the protective case provided with the order.

Note: Use care in removing your retainers. Do not use excessive force to bend or twist a retainer. Consult with your dental practitioner if your retainers are difficult to remove.

Daily care and maintenance of your Vivera retainers

1. Clean your retainers prior to each insertion. Use the Invisalign Aligner Cleaning System (recommended) or similar product or a soft bristle toothbrush using water or a small amount of toothpaste.
2. Rinse each retainer thoroughly with water after each cleaning.
3. Do not use denture cleaners to clean retainers or soak them in mouthwash. These products can damage the surface of the retainer, causing it to become dull and more visible.
4. Keep your retainers away from hot water, any heat source, or harsh chemicals.
5. Keep retainers out of reach of young children and pets.
6. Retainers may cause allergic reactions in some users. In such cases, discontinue use and contact your healthcare provider immediately.

Note: If your retainer becomes cracked, discontinue use and contact your dental practitioner for assistance.

Proper Oral Hygiene

1. Remove your retainers for eating and drinking, except when drinking water.
2. Brush and floss your teeth after each meal or snack prior to re-inserting your retainers. If you have any questions regarding your hygiene techniques, consult your dental practitioner.
3. Regular dental checkups and cleaning are recommended for the continued health of your teeth and gums.

Storing your Viverra Retainers

A retainer case is included with your retainers. When you are not wearing your retainers, we recommend that you always store them in this case in order to protect them from loss or damage.

Frequently asked questions and answers.

Q: Will wearing retainers be painful?

A: Some people experience slight tooth soreness. This is normal. This soreness will gradually go away in a couple of days. If it doesn't, promptly call your dental practitioner.

Q: Is it okay to drink hot or cold beverages while wearing Viverra retainers?

A: Except for water, we recommend that you do not eat or drink while wearing your retainers. This is to avoid formation of cavities or stains, and warping of the retainers that can occur if consuming extremely hot drinks.

Q: Can I chew gum while wearing Viverra retainers?

A: No, gum will stick to the retainers. We recommend removing your retainers for all meals and snacks.

Q: Will smoking or chewing tobacco stain the retainers?

A: We discourage tobacco use while wearing our retainers because of the possibility of retainer discoloration.

Q: Will wearing the Viverra retainers affect my speech?

A: It may temporarily affect your speech. You may have a slight lisp for a week or two. However, as your tongue gets used to having the retainers in your mouth, any lisp or minor speech impediment caused by the retainers should disappear.

Q: What if I lose or break my retainer?

A: In the event that your retainer is lost or broken, you should immediately contact your dental practitioner.

Q: What if I forget to wear my retainer?

A: Failure to wear the retainer for the required number of hours per day can affect the ability to maintain desired results.

Q: What should I do if my new retainer doesn't fit?

A: Retainers should fit comfortably and should not be forced into place. In the event of significant problems with retainer fit, discontinue use, and immediately contact your dental practitioner.

Note: If you have additional questions after reading this pamphlet, please consult your dental practitioner.

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